USO PREVISTO:

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GB

USE:

The medical devices "Micro screws and Micro pins for maxillofacial and dental surgical use", secure a membrane to the mandibular cortical bone and/or titanium meshes and/or small autologous / heterologous bone grafts.

DESCRIPTION

The Micro pins come in two lengths: 3 mm and 5 mm. The first are recommended for the majority of applications, while the 5 mm Micro pins provide greater stabilization in the presence of very soft bone. The special design of the Micro pin with crestal coils guarantees greater fixation of the membrane and simplifies its removal with the aid of a special screwdriver. Hexagonal engagement of 0.88 mm for easy removal. The principal region of use is the upper jawbone. The Micro screws are self-tapping and made from low profile titanium alloy with a Ø1.5 and a Ø2.5 mm head designed specifically to engage inner square tips (square engagement of 1.00 mm) which guarantees excellent stability of the Micro screws minimizing the risk of them freeing themselves during the application phase.

CAUTION:

The medical devices "Micro screws and Micro pins for surgical use" are designed to be used solely by certified surgeons and /or dental specialists and dental graduates who are certified to practice. Given the small size of micro-screws and micropins is necessary to use a medical device inserter that allows the easy grip, both during insertion both during the extraction phase so that there is no danger of committing technical errors during surgery and compromising the health status of the patient.

Excessive mechanical pressure or excessive pressure in the infusion of saline solution during the insertion / extraction can cause damage to the bone.

CAUTION

Before use, check the packaging is intact, otherwise the sterility is not guaranteed and it is necessary to sterilization prior to use as described in paragraph sterilization.

A micro screw / micro pin is disposable and cannot be reused, as this could cause the onset of infections or result in the failure to maintain the technical features.

MATERIALS:

The medical devices "Micro screws and Micro pins for surgical use" are produced entirely in grade 5 Titanium (Ti-6Al4V ELI).

STERILISATION:

The medical devices "Micro screws and Micro pins for surgical use" are supplied in sterile conditions (sterilization by Ethylene Oxide); this condition is ensured when the box is undamaged. If the box is damaged or there is partial use of the micro screws or micro pins contained in the rigid support, the specialist can re-sterilize the same following the steps below:

- Pack the rigid support with the micro screws or micro pins which are still unused (WARNING: the used micro screws or micro pins must be eliminated because they are contaminated and considered disposable - risk of onset of infections and/or loss of technical performance of the product) in an envelope (medical paper coupled with PET).
- Sterilize with steam in an autoclave at 121°C for 15 minutes or 134°C for 10 minutes.

MODE OF USE – MICRO SCREW:

Expose the part which will receive the micro screws and make the required holes with an irrigated drill at a low number of revs (500-700 revs-min). The pilot drill must have a diameter of 0.3-0.4 mm lower than the diameter of the micro screw. The micro screw must only be positioned and removed using the special tip.

MODE OF USE – MICRO PINS:

Expose the part which will receive the micro pin, engage the latter from the sterile blister using the special positioner and tap it gently into the preset site with the aid of a surgical hammer. To remove it use the special spanner with hexagonal engagement, unscrew the nail and remove using surgical tweezers.